

# LIFEPAK® 1000

Defibrillator



Highly configurable to adapt to your resuscitation protocol

Upgradeable platform to meet your resuscitation needs

The most rugged defibrillator ever built by Medtronic

Backed by our heritage of quality, reliability and service

Logical, natural interface with a large, intuitive screen

The nonrechargeable battery can deliver over 425 shocks

The LIFEPAK 1000 defibrillator is designed to be the most powerful basic life support (BLS) tool available. Fully customizable, it can adapt to the protocols of today and the future. cprMAX™ technology is highly flexible, with the most configurations available (as of January 2006) to accommodate your patient and CPR protocol requirements. Future patient-focused upgrades will help you provide the most advanced care in a way that complements how you work best. The 1000 is the most rugged defibrillator ever built by Medtronic, with an IP55 rating. ADAPTIV™ biphasic technology provides a full range of energy up to 360 joules.

With a large screen, graphics and text are clear and easy to read at a distance. The LCD screen can provide both graphical and ECG displays. A manual defibrillation override option is also available.

The advanced battery system in the LIFEPAK 1000 defibrillator provides more power for your patients. A visible and accessible battery gauge gives you the ability to assess battery readiness with the quick touch of a button. Our battery also performs advanced communications directly with the device so you always know the battery's condition.

The 1000 is compatible with other Medtronic products and services across the full spectrum of care, from the field to the hospital, including data download.

## DEFIBRILLATOR

All specifications are at 20°C unless otherwise specified.

**Waveform:** Biphasic truncated exponential with voltage and duration compensation for patient impedance\*

Energy Sequence: User configurable, 150 joules–360 joules. Default energy output settings are 200, 300, 360 joules. 360 joules for every shock thereafter.

Charge Time: With new, nonrechargeable battery pack; 200 joules in less than 7 seconds (360 joules in less than 12 seconds)


3-wire (Lead II) monitoring capability: (if ECG display option purchased) Requires purchase of 3-wire (Lead II) monitoring cable and LIFE-PATCH® electrodes

Device Software: Field upgradeable

Infant/Child Reduced Energy Defibrillation Electrodes: Reduces selected energy by a factor of 4. Intended for use only with children up to 8 years of age or 25kg (55 lbs).

Safety Classification: Internally powered equipment IEC 60601-1

**Electrical Protection:** Input protected against high voltage defibrillator pulses per IEC 60601-1

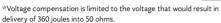

#### DEVICE SETTINGS

#### Modes:

- AED Provides operating capability for basic users
- Manual Provides operating capability for advanced users
- ECG Provides ECG display capability with 3-wire ECG cable
- Setup Allows user to configure the device
- Data Transfer Allows user to transfer patient data
- Auto Test Provides daily automatic tests of hardware and software

Controls: On/Off, Shock, Menu, Two (2) configurable soft keys

#### **User Defined Options:**

- $\bullet \ \, \textbf{Device ID} \textbf{Assigns unique identifier to particular device} \\$
- Energy Sequence User configurable from 150 to 360 ioules
- Flexible Energy Increases only after a lower energy was unsuccessful
- Auto Analyze User can configure device to auto analyze, auto analyze after first shock, or prompt user to push analyze key before each analysis period
- CPR Time (post shock or after no shock advised) User configurable 15, 30, 45, 60, 90, 120, 180 seconds
- · Device Date/Time
- Voice Prompt Volume Allows user to change speaker volume
- ECG Display (if option purchased) Turns display on/off for AED mode

- Motion Detection User defined On/Off (default Off)
- Service Alert Audio alarm if the device needs servicing. Configurable on/off.
- Manual Access (if ECG display option purchased)
   Devices configured with an ECG display may be set up to allow user to initiate a charge and shock without analysis

## cprMAX Technology Settings:

- Initial CPR User defined time for CPR after first analysis regardless of analysis decision. Can be set to OFF, 15, 30, 45, 60, 90, 120 and 180 seconds.
- Preshock CPR Allows for CPR while device is charging
- Confirmation Analysis Confirms shockable rhythm after completion of Initial CPR or Preshock CPR periods and prior to Push to Shock prompt
- Stacked Shocks (ON/OFF) When Off, allows for provision of CPR after each shock
- Pulse Check (Always, After Every NSA, Never) Allows device to prompt for a pulse check either after each shock, after every NSA, or never prompt for a pulse check (default Never)

#### DISPLAY

Backlit LCD displays number of shocks delivered, elapsed time, text and graphics of heart rhythm and optional ECG

Size: 120 mm (4.7 in) x 89 mm (3.5 in)

Frequency Response: 0.55 Hz to 21 Hz (-3 dB), nominal ECG option:

- Waveform Sweep Speed 25 mm/sec for ECG, nominal
- Waveform Viewing Time Minimum 4 seconds
- Waveform Amplitude 1 cm/mV, nominal
- Heart Rate 20 to 300 BPM digital display, Display "---" if heart rate is less than 20 bpm. Heart symbol flashes for each QRS detection.

ECG information is received from the adult and Infant/Child electrodes in anterior-lateral or anterior-posterior positions. A 3-wire cable can be used for ECG monitoring (Lead II).

#### ENVIRONMENTAL

One Hour Operating Temperature (from room temperature to temperature extreme, one hour duration): -20 to  $60^{\circ}$ C (-4 to +140°F)

Operating Temperature: 0 to 50  $^{\circ}$  C (32 to 122  $^{\circ}$  F)

Storage Temperature: -30 to 60  $^{\circ}$  C (-22 to +144  $^{\circ}$ F) with battery and electrodes (maximum exposure limited to 7 days)

**Atmospheric Pressure:** 575 hPa to 1060 hPa (4572 to -382 meters; 15,000 to -1253 feet)

Relative Humidity: 5 to 95% (non-condensing)

**Dust/Water Resistance:** IP55 with battery and REDI-PAK™ electrodes installed (IEC 60529/EN 60529)

**Bump:** 15 g, 1000 bumps (IEC 600-68-2-29)

Shock: 40 g peak, 15 - 23 ms, 45 Hz cross over frequency

**Drop:** 1 meter drop on each corner, edge and surface (MIL-STD-810F, 516.5, Procedure IV)

**Vibration:** Random vibration test - MIL-STD-810F, Method 514.5, Category 20; Ground vehicle 3.15 g rms 1 hour per axis

#### EMI

- Radiated IEC 60601-2-4, IEC60601-1-2, CISPR 11 Class B Group 1
- Immunity IEC 60601-2-4, IEC 60601-1-2; IEC 61000-4-2 (Level 4), IEC 61000-4-3, IEC 61000-4-6, IEC 61000-4-8

## EVENT DOCUMENTATION AND COMMUNICATION

**Memory Capacity:** Dual patient storage. Minimum 40 minutes ECG for current patient. Summarized data for previous patient.

Report Types: Continuous ECG, summary (critical resuscitation events and associated ECG waveforms), event log report (report of time stamped entries reflecting operator and device activity), test log report (self test activity report)

Capacity: Minimum 100 time stamped event log entries

Data Review: CODE-STAT™ Suite 6.1 Medical Informatics System, LIFENET® DT Express 2.1 Information Management System

**Communications:** Infrared wireless transfer to personal computer

### BATTERY AND READINESS DISPLAY

Note: See operating instructions for information on battery care

Primary Battery (nonrechargeable battery with status indicator):

- Type Lithium Manganese Dioxide (Li/MnO<sub>2</sub>), 12.0V, 4.5 amp-hours
- Capacity Typically will provide 440 200 joule shocks or 1030 minutes of operating time with a new battery (370 200 joule shocks or 900 minutes of operating time at 0°C)
- Weight 0.45 Kg (1.0 lb)
- Shelf Life After the battery is stored for 5 years at 20°C to 30°C, the device will provide 48 months of standby life
- Standby life (assuming daily tests only) A new battery provides device power for 5 years
- Low battery indication At least 30 shocks or 75 minutes of operating time remain when low battery is first indicated

# PHYSICAL CHARACTERISTICS

Height: 8.7 cm (3.4 in) Width: 23.4 cm (9.2 in) Depth: 27.7 cm (10.9 in)

**Weight:** 3.2 kg (7.1 lbs) with one set of REDI-PAK electrodes and one nonrechargeable battery

For further information please call Medtronic at 1.800.442.1142 or visit www.medtronic-ers.com

Medtronic Emergency Response Systems 11811 Willows Road NE P. O. Box 97006 Redmond, WA 98073-9706 USA Tel: 425,867,4000 Fax: 425,867,4121 www.medtronic-ers.com

Tolochenaz, Switzerland Tel: 41.21.802.7000 Fax: 41.21.802.7900

Switzerland Tolochenaz, Switzerland Tel: 41.21.803.8000 Fax: 41.21.803.8099 Canada Mississauga, Ontario Tel: 905.826.6020 Fax: 905.826.6620

United Kingdom, Ireland Watford, Great Britain Tel: 44.1923.212.213 Fax: 44.1923.241.004 France Boulogne-Billancourt, France

Fax: 33.1.55.38.1800 Germany Dusseldorf, Germany Tel: 49.211.529.30 Fax: 49.211.529.31.00

Tel: 33 1 55 38 1700

Austria Vienna, Austria Tel: 43.1.240.44.160 Fax: 43.1.240.44.600

Italy Milan, Italy Tel: 39.02.24137.1 Fax: 39.02.24138.1

Netherlands Heerlen, The Netherlands Tel: 31.45.566.8000 Fax: 31.45.566.8668

**Spain**Madrid, Spain
Tel: 34.91.625.04.00
Fax: 34.91.650.74.10

Scandinavia Järfälla, Sweden Tel: 46.8.52.22.00.00 Fax: 46.8.52.22.00.50

Asia Pacific Hong Kong, S.A.R., PRC Tel: 852.2891.4456 Fax: 842.2891.6830

Latin America & Caribbean Miami, Florida USA Tel: 305.500.9328 Fax: 786.709.4244

Middle East Beirut, Lebanon Tel: 961.1.370.670 Fax: 961.1.364.164 Hungary Budapest, Hungary Tel: 36.1.214.2228 Fax: 36.1.214.2230

Tel: 48.22.465.69.00 Fax: 48.22.465.69.17 Czech Republic Prague, Czech Republic Tel: 420.2.2017.2277

Fax: 420.2.2056.1617

People's Republic of China
Shanghai, China
Tel: 86.21.50800998

South Africa Bedfordview, South Africa Tel: 27.11.677.4800 Fax: 27.11.616.1104

**Japan** Kawasaki, Kanagawa, Japan Tel: 81.44.540.6502 Fax: 81.44.540.6170

Australia Sydney, Australia Tel: 61.2.9879.5999 Fax: 61.2.9879.5100

Israel Even-Yehuda, Israel Tel: 972.9.891.2223 Fax: 972.9.891.9330

LIFEPAK, LIFE-PATCH and LIFENET are registered trademarks of Medtronic Emergency Response Systems, Inc. cprMAX, CODE-STAT, ADAPTIV and REDI-PAK are trademarks of Medtronic Emergency Response Systems, Inc. Medtronic is a registered trademark of Medtronic, Inc. ©2006 Medtronic Emergency Response Systems, Inc. MIN 3206045-002 / CAT. 26500-002037

